


## XLR8 Quick Application Reference Guide

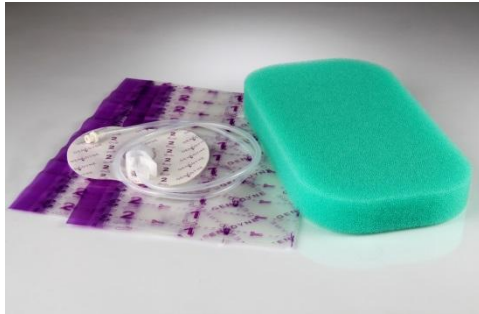

XLR8 Foam Kit. Choose appropriate size for wound.



Clean the wound according to agency protocol. Protect wound edges with XLR8 drape.



Cut foam to appropriate size of the wound. Place foam into the wound. Avoid cutting the foam over the wound. Do not over pack



Cover foam with XLR8 drape. Peel layers 1,2,3. Remove purple handlers



Cut a hole in the covering drape approximately the size of a quarter. Remove paper backing (number 1) from the Port pad. Place over the hole. Peel number 2. Remove handlers.



Connect the Port pad to the canister tubing with the secure luer lock.



Initiate therapy at prescribed pressure by turning unit on.



Foam will collapse down and target pressure reached.

For more information contact PMR, your Genadyne representative, at 1-800-633-6978